


Product information

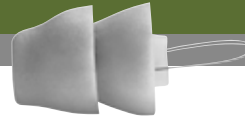

Lyric2 is an extended wear hearing aid that can be worn in the ear 24 hours a day, 7 days a week for up to 120 days. Due to the deep placement inside the ear canal, Lyric2 is truly 100% invisible. Lyric2 is very easy to operate with no batteries to change, no ongoing maintenance required and no daily insertion or removal.

A Lyric2 trained hearing care professional determines if a person can wear Lyric2 based on an assessment of hearing loss, ear size and shape, medical conditions and lifestyle. Lyric2 is appropriate for people with mild to moderately-severe hearing loss (not suitable for sloping hearing losses with more than 30 dB per octave).

Lyric2 takes advantage of the deep placement in the ear for natural sound quality and the natural directivity of the pinna.


It uses an analog, digitally programmable WDRC circuit with very low power consumption.

The SoundLync allows the wearer to adjust the volume, turn the hearing aids on and off and place them in sleep mode.

Lyric2 is designed for single insertion. Do not reuse a hearing aid once removed from the ear.


Technical description

- 1 Insertion handle
- 2 Removal loop
- 3 Microphone protection
- 4 Microphone and signal processor
- 5 Lateral seal
- 6 Medial seal
- 7 Receiver
- 8 Battery
- 9 "This side up" indication

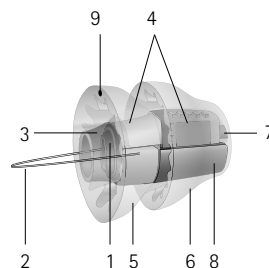


SoundLync

- 1 Keychain housing
- 2 Magnet
- 3 Removal tool
- 4 Cap

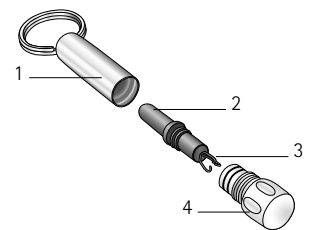

Fitting range

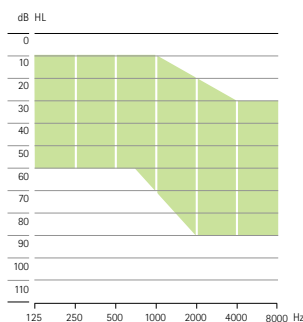

What is the difference between Lyric and Lyric2?

The main difference relates to the size of the device. Lyric2 is shorter and smaller and therefore fits more ear canals. In addition, newly shaped seals ensure enhanced wearing comfort.

Can I switch a running subscription from Lyric to Lyric2?

Yes. Lyric wearers should always benefit from the latest technology available. Phonak recommends upgrading existing Lyric wearers to Lyric2.

Is resizing required to fit Lyric2?

Yes. The shape, length and size of Lyric2 are different. You will need to resize your existing Lyric wearers.

What is Lyric2 and what are the differences to traditional hearing aids?

Lyric2 is an extended wear hearing aid that can be worn in the ear 24/7 even while sleeping, exercising and showering for up to 120 days. It is worn deep inside the ear canal and therefore completely invisible. It requires no handling or maintenance by the wearer. It is a completely hassle-free solution. Lyric2 comes in 5 different sizes and can be fit instantly.

Who is a Lyric2 candidate?

A Lyric2 trained hearing care professional will evaluate candidacy based on four main criteria:

1. hearing loss within the mild to moderately-severe fitting range
2. medical history and condition
3. ear geometry and anatomical features
4. lifestyle (motivation level, limitation to swim underwater or skydive / scuba dive when wearing a device)

How do I fit Lyric2?

Lyric2 is a deeply placed hearing aid residing 4 mm from the tympanic membrane. To ensure a comfortable and accurate fitting, each hearing care professional will be trained to use specifically designed measurement and placement tools. First the length and diameter of the ear canal is measured to determine the right insertion depth and device size. The hearing aid is then programmed and placed in the canal. Fine-tuning can then be performed, with the hearing aid in situ, via the Lyric programming wand.

Is Lyric2 waterproof?

Lyric2 is water resistant which means that swimming is not possible unless you wear a custom made earplug to protect Lyric2 from water exposure. Showering or walking in the rain however is possible. You cannot sky dive or scuba dive with Lyric2.

What are the advantages of the deep placing?

Placing Lyric2 deep in the ear canal offers the following advantages: improved directionality and localization, reduced occlusion and feedback, high frequency gain, reduced wind noise, less distortion due to less gain required, normal telephone use, maximum comfort with 100% invisibility, ease of use with no daily insertion or removal and no need to change the battery.

How is Lyric2 programmed?

Phonak Lyric2 is wirelessly programmed via Phonak Target and a programming wand. The fitting parameters are transferred to the hearing aid using a magnetic impulse. For programming, the wand has to be placed close to the microphone protection of the hearing aid. The communication is one-way; Lyric2 can be programmed but not read-out.

Can the wearer remove the hearing aid?

It is recommended to have the hearing aid removed by the Lyric2 trained hearing care professional. If necessary the wearer can remove Lyric2 with a provided removal tool.

Can the wearer change hearing aid settings?

Lyric2 is wirelessly programmed for the individual hearing loss of the wearer. The wearer is able to change volume and switch the hearing aid on, off or into sleep mode with a remote control magnet called SoundLync. The SoundLync also functions as a removal tool if the wearer wishes to remove his/her Lyric2 hearing aid(s).

What is SLEEP mode?

SLEEP mode is a setting for acoustic transparency. In this setting, Lyric2 compensates the damping caused by the hearing aid itself. For the wearer, it is as if they are not wearing a hearing aid.

How is the battery changed?

Lyric2 is a disposable hearing aid designed for single insertion. Once the battery is depleted, the hearing aid is removed and then replaced with a new Lyric2 hearing aid.

How long does Lyric2 last?

Lyric2 may be worn for up to 120 days or until it ceases to function.